

Thompson Retractor

Adjustable Scope Grip (#33181)

Free Up Staff


Hold various sized scopes with the Adjustable Scope Grip when used in combination with Thompson laparoscopic holders. The Adjustable Scope Grip easily adjusts to fit 5-6mm and 10-11mm scopes and holds scope firmly in position to free up staff.

BENEFITS

- Adjustable to hold various sized scopes
- Firmly holds and protects scope shaft
- Quickly connects to Thompson laparoscopic holders

"Thompson Surgical provides the most hassle-free, versatile retractor."

- Michael Nizzi, DO, FACOS

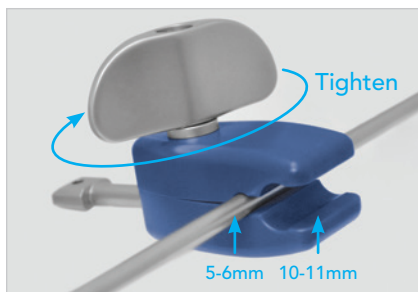

Step 1

Attach to scope shaft (5-6 or 10-11mm), rotating knob clockwise to tighten.

NOTE: DO NOT OVERTIGHTEN



Step 2


Insert into laparoscopic holder distal end.



Step 3

Rotate hanging knob clockwise to secure and hold scope hands free.

 **ORDERING INFO:**
Adjustable Scope Grip 5-6mm / 10-11mm #33181


 **NOTICE:**
Recommended for use with the Articulating Arm or Flexible Scope Holder plus table mount.

 **Thompson**
SURGICAL INSTRUMENTS
10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

For a Free Trial Call **1-800-227-7543** *

* Free trial valid for U.S. customers only. Customers outside the U.S.A., please call +1-231-922-0177 for availability.

© 2020 Thompson Surgical Instruments, Inc.

© S-Lock® ®, PLA®, and the Thompson "T Circle" logomark  are Registered Trademarks of Thompson Surgical Instruments, Inc.

Patents: trpat.com

Other patents pending.

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
THE NETHERLANDS

 0297

Rev G
050823
asgss

**Made In
USA**