
Thompson Retractor

Uncompromised Exposure™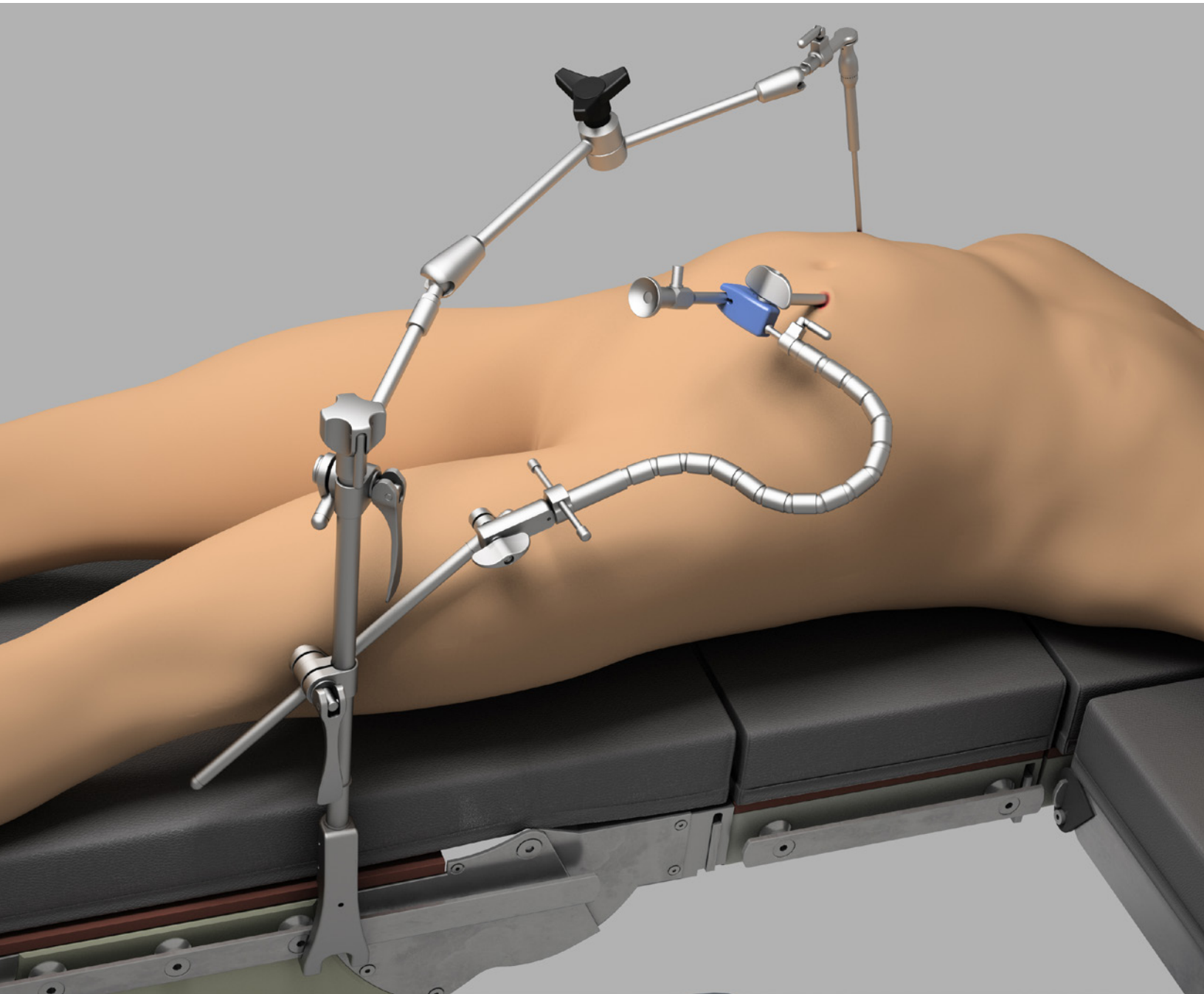

Laparoscopic User Manual:
SET UP INSTRUCTIONS FOR THOMPSON LAPAROSCOPIC PRODUCTS

Table of Contents

Warnings + Cautions	3
Elite Rail Clamp Set Up	4
Power Rail Clamp Set Up	5
Holding Arms	6
Laparoscopic Articulating Arm Set Up	6
Flexible Laparoscopic Holder Set Up.....	6
Rigid Laparoscopic Holder Set Up	6
Adding Attachments	6
Attachments	7
Adjustable Scope Grip	7
S-Lock Nathanson Hooks	7
Scope Clip	7
Duo Head.....	7
Scope Grip + Bushings	7

NOTE

As we continually strive to provide the best products possible, some of the images in this user manual may appear slightly different from the product received.

CAUTION: Avoid compressing the patient's body with frame components to prevent nerve damage.

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
Elite Rail Clamp Set Up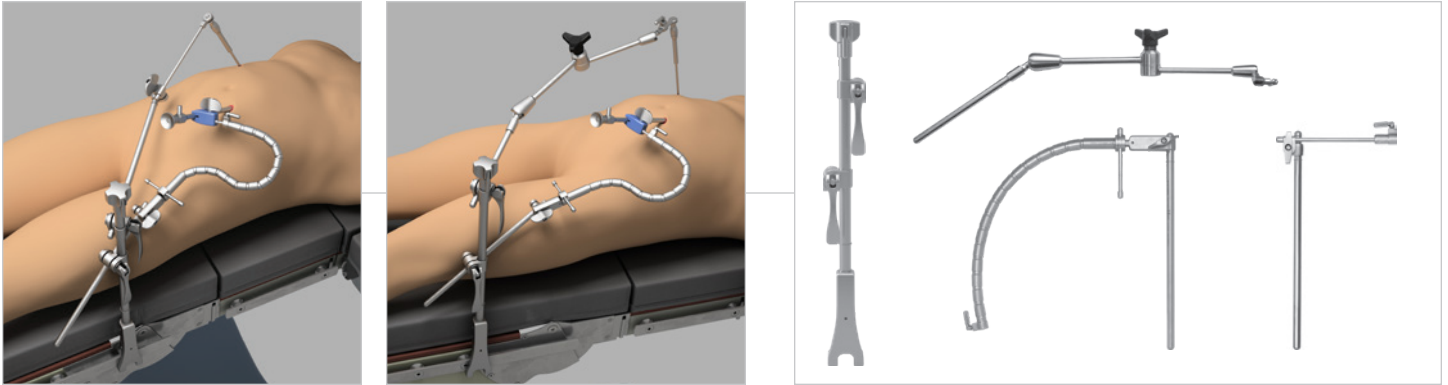

STEP 1: RAIL CLAMP

Place Elite Rail Clamp onto the rail over the sterile drape and turn top knob clockwise to tighten.


STEP 2: HOLDING ARM

Insert holding arm of choice into the joint on the Elite Rail Clamp, position, and flip cam joint handle to lock. Repeat with a second holding arm and second Elite Rail Clamp joint if desired.


Power Rail Clamp Set Up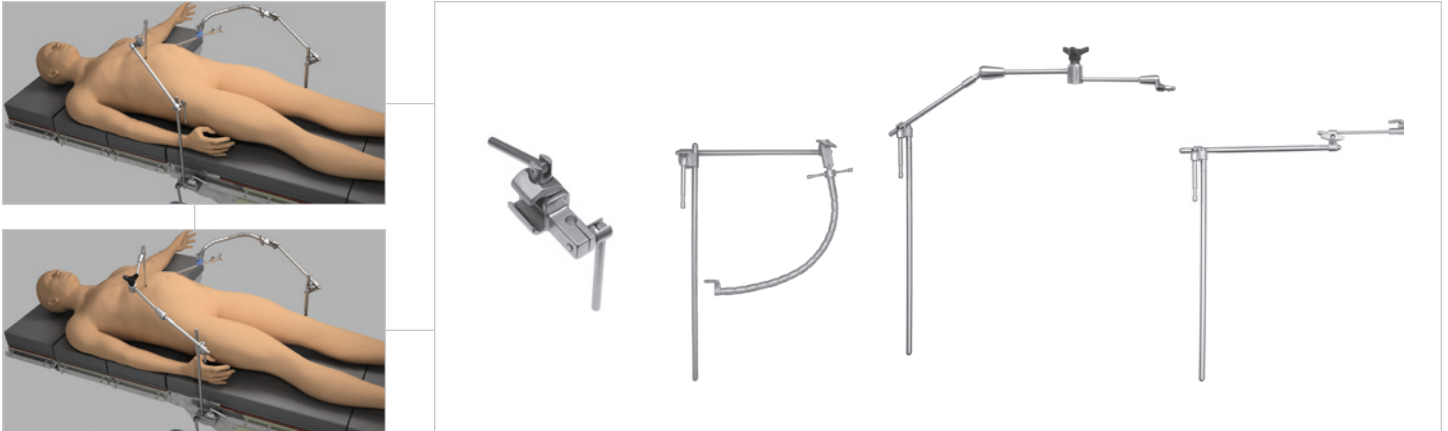

STEP 1: RAIL CLAMP

Place Power Rail Clamp onto the rail over the sterile drape and turn top knob clockwise to tighten.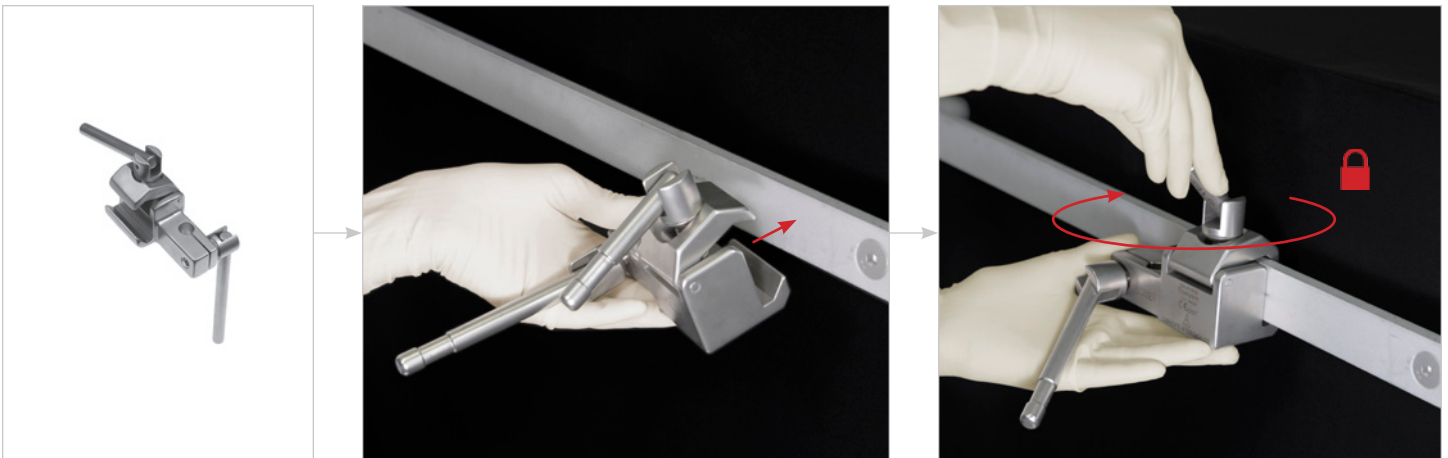

STEP 2: HOLDING ARM


Insert holding arm of choice into the opening on the Power Rail Clamp, adjust height of arm, and turn bottom knob clockwise to tighten. Position proximal horizontal portion of arm and flip the handle on the cam joint to secure in place. Repeat with a second Power Rail Clamp and holding arm on opposite side of table if desired.

Holding Arms (Elite and Power Rail Clamp)


LAPAROSCOPIC ARTICULATING ARMS

Once secured to the rail clamp, release arm tension by turning the black knob counter-clockwise. Adjust position of arm, ensuring it will allow the scope or instrument to reach the incision, and turn the black knob clockwise to secure.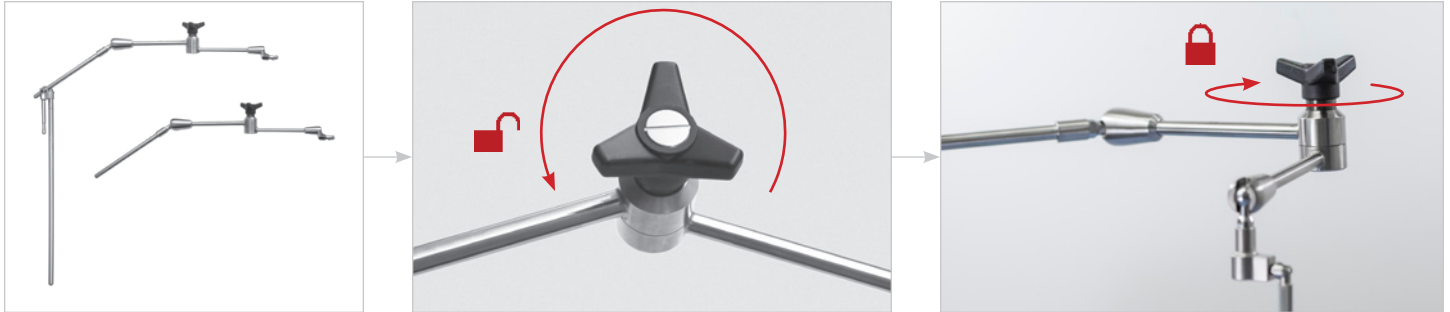


FLEXIBLE LAPAROSCOPIC HOLDERS

Once secured to the rail clamp, adjust position of arm to ensure it will allow the scope or instrument to reach the incision and turn the universal joint knob clockwise to secure. Adjust tension of the gooseneck portion by turning the hanging handle counter-clockwise to loosen, clockwise to add rigidity.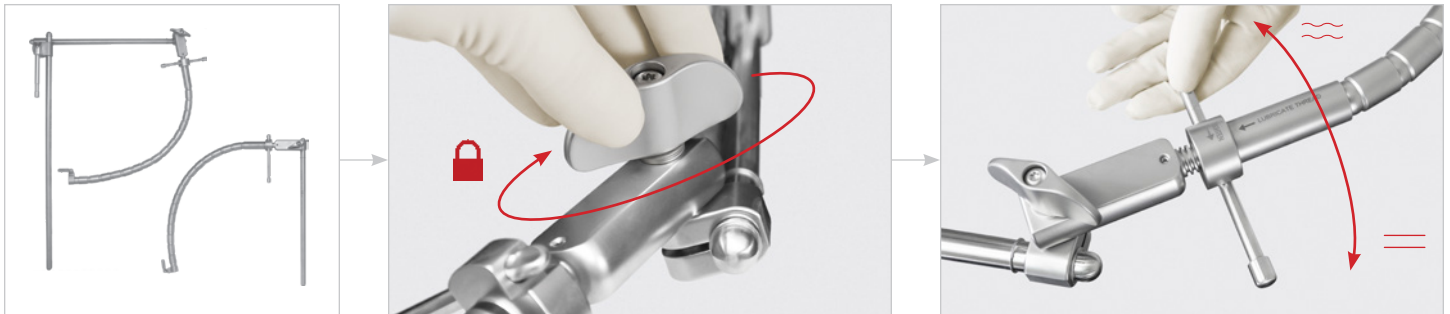


RIGID LAPAROSCOPIC HOLDERS

Once secured to the rail clamp, adjust position of arm to ensure it will allow the scope or instrument to reach the incision and turn the universal joint knob clockwise to secure.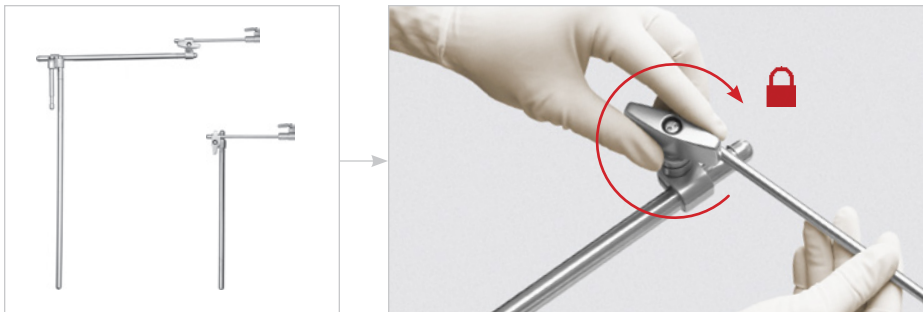

ADDING ATTACHMENTS

Insert attachment of choice into the universal port on the distal end of holding arm. Turn the knurled knob clockwise to secure.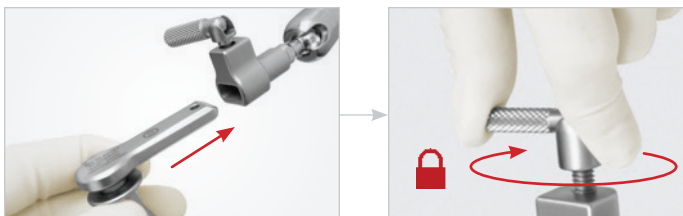


NOTE

If using Rigid Laparoscopic Holder with integrated Duo Head or S-Lock® Nathanson Adapter, skip this step.

Attachments

ADJUSTABLE SCOPE GRIP

After securing Adjustable Scope Grip to holding arm, turn knob counter-clockwise to open, insert scope shaft (5 - 6mm or 10 - 11mm), and turn knob clockwise to tighten. **NOTE: DO NOT OVERTIGHTEN**

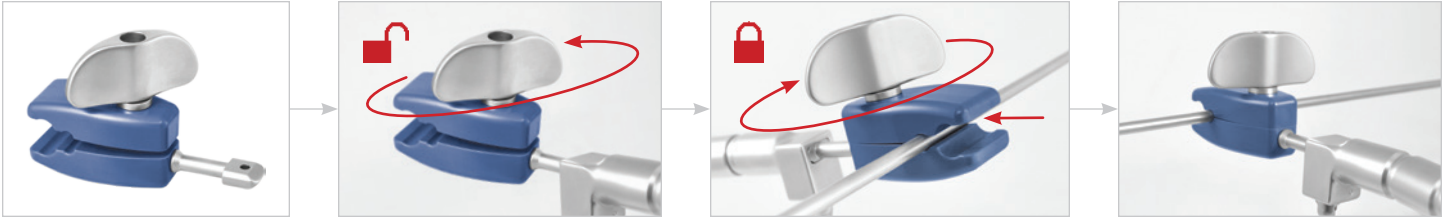S-LOCK NATHANSON ADAPTER + HOOKS (OR RIGID HOLDER WITH INTEGRATED S-LOCK NATHANSON ADAPTER)

After securing S-Lock Nathanson Adapter to the holding arm, Pull back plunger fully and insert S-Lock Nathanson Hook nipple, align serrations, and release.


SCOPE CLIP

After securing Scope Clip to the holding arm, insert scope into Scope Clip and position. Scope may be easily adjusted for in and out panning.


DUO HEAD (OR RIGID HOLDER WITH INTEGRATED DUO HEAD)

After securing Duo Head to holding arm, turn knob to open, insert 3mm, 5mm, or 10mm laparoscopic instrument into corresponding sized opening of Duo Head, and turn knob clockwise to tighten.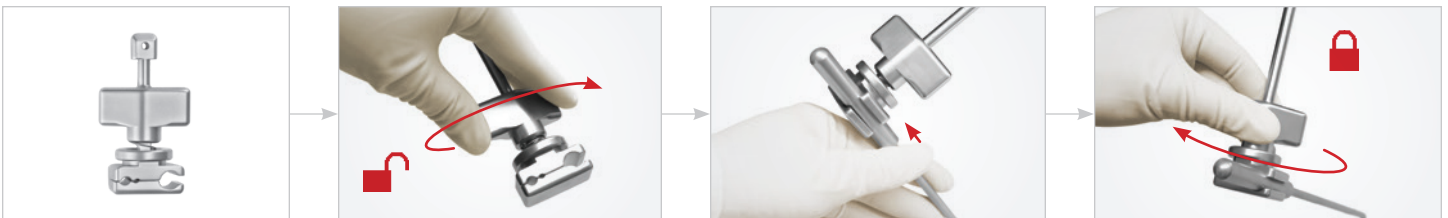

SCOPE GRIP AND BUSHINGS

After securing Scope Grip to holding arm, turn knob to open. Insert scope into bushing. Insert scope and bushing into Scope Grip. Position scope and turn the joint to tighten, securing in place.

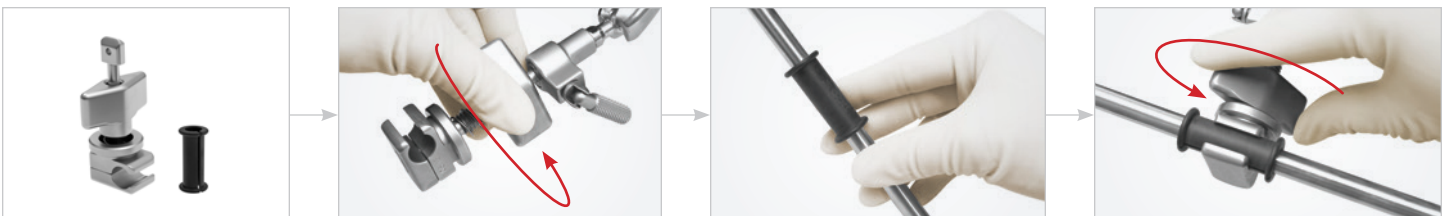For a Free Trial Call Today*

1.800.227.7543

Rev I
011123
dc lapum

Uncompromised Exposure

VISIT US ONLINE: THOMPSONSURGICAL.COM

Thompson
SURGICAL INSTRUMENTS

10341 East Cherry Bend Road
Traverse City, Michigan 49684
phone: 231.922.0177
fax: 231.922.0174
thompsonsurgical.com

EC REP

Emergo Europe

Westervoortsedijk 60
6827 AT Arnhem
THE NETHERLANDS

* Free trial valid for U.S. customers only. Customers outside U.S. please call +1-231-922-0177 for availability.

© 2020 Thompson Surgical Instruments, Inc. Printed in the U.S.A.

® S-Lock®, PLA®, and the "T Circle" logomark are Registered Trademarks of Thompson Surgical Instruments, Inc. Patents: trpat.com
Other patents pending.

Symbol Legend:

 Manufacturer	 Authorized EC Rep	 CE Marking	 Warnings / Precautions	 Non-Sterile
---	--	---	---	--

CE 0297

Made In
USA